

Trouble Shooting Chart

V1.0

(C) 2011 Infantrust. Not for public distribution.

Respisense is very easy to use and not prone to false alarms. When problems do occur, follow these guidelines to assist customers.

The monitor must be close to baby's navel, slightly to one side.

If baby sleeps on his side, the monitor must be slightly towards the side that baby sleeps on, between baby's navel and the bed.

Symptom	Possible Causes	Solution
Movement not detected.	Not making proper contact with tummy: - Baby sleeps on his side. - Diaper not secure enough. - Wrong position on diaper. - Diaper too small. - Diaper is very wet.	Place the monitor more to the side on which the baby sleeps. Diaper must fit snugly, not too loose. Back of clip must make contact with baby's tummy at all times. The monitor must be close to baby's navel. If the diaper is a size too small the monitor will not to make proper contact with the tummy. Change diaper when wet. With boys, make sure that urine will be upsorbed more towards the bottom of the diaper, rather than the top.
Blue light flashes.	Not making proper contact with tummy.	Tighten diaper.
Red light flashes when on/off button is pressed.	Battery depleted.	Replace the battery.
Monitor will not switch on.	Battery completely depleted.	Replace the battery.
Broken clip.	Clip forced over too thick clothing. Too thick cloth diaper is being used, or waistband of disposable diaper is folded.	Use diaper no thicker than a typical disposable one. Do not fold waistband double.



Soft flap folded over, but not the waistband.